

Zephyr™ HxM™ Smart

USER GUIDE



PACKAGE CONTENTS

Congratulations on purchasing a Zephyr™ HxM™ Smart Heart Rate Monitor. This package contains the following:

1. Zephyr™ HxM™ Smart, Bluetooth™ Smart enabled heart rate monitor
2. Zephyr™ Smart Fabric sensing strap.

COMPATIBLE PHONES

The Zephyr™ HxM™ Smart Heart Rate Monitor utilizes Bluetooth 4.0 also known as Bluetooth™ Smart or Bluetooth™ Low Energy.

Works with iPhone 4s, iPhone 5 and certain Android based Smartphones.

Check the web for phone compatibility
www.ZephyrAnywhere.com/ConsumerFitness



FEATURES

- Heart Rate and R-R data
- Accessibility to features will depend on the app being used to receive transmitted data.


APPS

Zephyr™ HxM™ Smart Heart Rate Monitor works with a wide variety of Smartphone apps. For the latest list of compatible apps, to go www.ZephyrAnywhere.com/ConsumerFitness or the Apple Apps Store or Google Play.


OPERATING INSTRUCTIONS

Wearing your device

- Snap your HxM™ Smart device to your chest strap

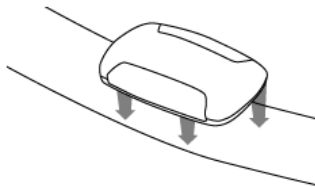

- Moisten the sensor pads on the inside of the chest strap and put it on

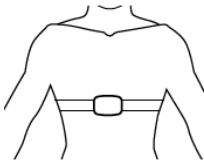


DOWNLOAD THE APP AND CONNECT YOUR HxM™ SMART

- Select, download and install the app on your Smartphone.



- Make sure you are wearing your HxM™ Smart device so that it is turned on. A green LED will flash every 15 seconds.
- Start your app and connect it to your HxM™ Smart device

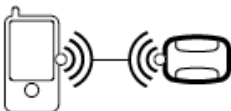

SPECIFICATIONS

HR Range:	25 – 240 BPM
Battery Life:	150 Hours
Transmit Range:	10m
Frequency:	2.4 – 2.4835GHz
Garment Washes:	50
Operating Limits:	
Temperature:	-10 – 50°C
Humidity:	5 – 95%

GUIDE TO SAFE & EFFECTIVE USE

- Please consult your physician before exercising with this device
- Please do not use this device around other medical equipment without permission
- Please do not use this device in places where wireless devices are prohibited, such as on aircraft

CARE AND MAINTENANCE

- If required, wipe clean with a moist cloth
- Do not leave in direct sunlight
- Please do not expose to chemicals
- Can be immersed in water (1m) for short periods (20 min)
- Keep away from fire or other heat sources

STRAP



- Wash after every 30 days of use
- Machine wash cool (<math><40^{\circ}\text{C}</math>)
- Delicate setting
- Hang (NOT tumble) dry
- Do not bleach or iron
- Do not dry clean

WARRANTY

- 1 Zephyr provides a limited 12 month warranty for the BioHarness 3 product. Zephyr warrants to the original purchaser of this product that it will replace, repair or refund the cost of this product or any part proven to be defective in material or workmanship.. In no event shall Zephyr be liable for direct, incidental or consequential damages arising out of use of this product, and any recovery is limited to the purchase price.
- 2 Warranty claims will be initiated by emailing warranty@zephyr-technology.com
- 3 During the warranty period, if this product does not work properly under normal use and care conditions, and if this is caused by a design, material or workmanship defect, Zephyr will decide to replace or repair the product at their discretion according to the following conditions. If a returned product is found not within the range of the warranty, Zephyr retains the right to charge a handling fee.
 - a. The warrant is effective only when proof of purchase can be provided.
 - b. If Zephyr repairs or replaces the product, the repaired or replaced product will continue to

be covered for the balance of the warranty period.

- c. If it is deemed that the device has not yet been used or maintained as recommended, and is considered reasonable, the warranty will not apply to this product.
- d. If the product has been altered in any way by non-Zephyr authorized personnel, the warranty will not apply to this product.
- e. A malfunction or damage caused by use of the Zephyr product with other products will not be covered by the warranty.



This device has been designed to conform to the following:

- FCC Part 15
- RTTE Directive 1999/4/EC
- This product complies with RoHS 2002/95/EC
- This device contains transmitter module
FCC ID: T7V1315
- This device complies with Part 15 of the FCC Rules.
Operation is subject to the following conditions:
 - This device may not cause harmful interference, and
 - This device must accept any interference received,
including interference that may cause undesired
operation.

Note: The manufacturer is not responsible for any radio or TV interference caused by unauthorized modifications to this equipment. Such modifications could void the user's authority to operate the equipment.

Contact:

Zephyr Technology
1 Annapolis St
Suite 200
Annapolis
MD 21401
USA

Tel: +1 (443) 569-3603

Fax: +1 (443) 926-9402

Email: support@zephyrtech.zendesk.com
info@zephyranywhere.com
sales@zephyranywhere.com

Web: www.zephyranywhere.com