



Zephyr USB Driver

Manual Installation Instructions

1. Introduction

Use these instructions to manually install Zephyr USB Driver version 4.40s (digitally signed version).

If your PC already has Zephyr USB Driver 4.40, then these must first be removed manually:

1. Disconnect any Zephyr USB devices from your PC.
2. Go to:

***Windows > Start > Control Panel > Add or Remove Programs (XP) or
Windows > Start > Control Panel > Programs & Features (Vista)***



Scroll down and locate any previous version of the Zephyr USB Driver

4. Use the ***Change/Remove*** (XP) or ***Uninstall*** (Vista) button to remove the old driver files.

2. Installing Drivers

1. All required files are delivered in a directory titled 'Zephyr USB Driver' which may be provided as either a zipped email attachment, on a CD, or as a web download.
2. Browse the directory and locate the '*ZephyrDeviceInstaller*' executable



3. Double-click the executable file



4. When the Driver files have installed, click OK to restart your PC

